

# X803G EQUINE TECHNIQUE CHART \*

**X-ray Unit: MinXray, Inc. Model X803G, & X903G**  
**Output Setting: 80kVp / 20mA**  
**Focal Film Distance: 24-26 Inches**  
**Screen/Film Combination: 400 Speed Rare Earth**

ANATOMY	VIEW	TIMER SETTING, seconds
CARPUS	ALL	.06 - .08
FETLOCK	ALL	.06 - .08
METACARPALS	ALL	.06 - .08
METATARSALS	ALL	.06 - .08
RADIUS	AP	.08 - .1
RADIUS	LATERAL	.08
TARSUS	AP	.1 - .15
TARSUS	ALL OTHERS	.1
STIFLE (1)	PA	.2 - .4
STIFLE	ALL OTHERS	.15 - .2
WITHERS	DORSAL	.08
WITHERS	VENTRAL	.15
NAVICULAR	AP 65° CONE DOWN	.1
THIRD PHALANX	AP 65°	.04
PROXIMAL BORDER	AP 45°	.06 - .08
LATERAL		.06 - .08
FLEXOS (2)		.08
SINUS/DENTAL	LATERAL	.08 - .1
SINUS/DENTAL	OBLIQUES	.06 - .08
SINUS/DENTAL	DORSOVENTRAL	.2 - .3

(1) SHORTEN FFD ON STIFLES TO 20 - 22 INCHES

(1) SHORTEN FFD ON FLEXOR TO APPROXIMATELY 20 INCHES

3611 Commercial Avenue  
 Northbrook, Illinois 60062-1822, U.S.A.  
 (847) 564-0323  
 Toll Free: 1 800 221-2245  
 FAX: (847) 564-9040